



My Information

Please provide the following information and bring this brochure with you to your next appointment:

My Apple device is:

My Apple ID is: _____

User Name: _____

Password: _____



My other streaming devices:





Distributed by:

Costco Wholesale UK Ltd

Costco Online UK Ltd
Hartspring Lane
Watford, Herts
WD25 8JS
United Kingdom
01923 213113
www.costco.co.uk

Costco Wholesale Australia Pty Ltd

17-21 Parramatta Road
Lidcombe NSW 2141
Australia
www.costco.com.au

Kirkland Signature 6.0 Hearing Instruments are compatible with iPhone 6, iPhone 6 Plus, iPhone 5s, iPhone 5c, iPhone 5, iPad Air 2, iPad Air, iPad (4th generation), iPad mini 3, iPad mini 2, iPad mini and iPod touch (5th generation) using iOS 7.X or later. Apple, the Apple logo, iPhone, iPad, iPod touch and iTunes are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc.

M201184-GB-15.02-RevA

KIRKLAND
Signature™

Get a good start
with your Kirkland
Signature 6.0
Hearing Instruments
with Made for
iPhone functionality



Made for
iPod iPhone iPad

Welcome to a new generation of hearing aids

Your Costco Hearing Aid Center professional has placed your order. You will soon be ready to experience our Made for iPhone (MFi) hearing instruments, including the ability to stream sound directly from your device to your hearing instruments. In order to prepare for your next appointment, you are kindly asked to download the Choice app.



How to download the Choice app:

Step 1: Make sure your Apple device is compatible with your Made for iPhone hearing aids.

Made for iPhone hearing aids are compatible with the following Apple devices:


- iPhone® 6, iPhone 6 Plus, iPhone 5s, iPhone 5c, iPhone 5
- iPad Air® 2, iPad Air, iPad® (4th generation), iPad mini 3, iPad mini 2, iPad mini with Retina display, iPad mini
- iPod touch® (5th generation)

Using iOS 7.X or later.

Step 2: Make sure your Apple device has iOS 7.X or higher.

1. Check the iOS software on your Apple device to ensure the latest version is installed. Tap Settings> General> Software Update
2. If a software update is available, tap Install

Note: You must be connected to Wi-Fi to update iOS.

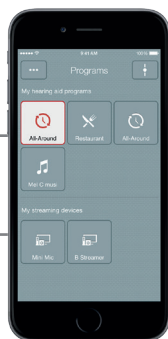

Step 3: Download the Choice app.

1. Tap the App Store icon. Select **iPhone only** setting. Please note that this will allow you to download to an iPad and an iPod touch too. This setting is only to locate the Choice app.
2. Type **Kirkland Signature Choice** in the search field.
3. Once the Choice app screen is displayed, tap **Free** (It will then change to **Install**).
4. Tap **Install** to load the app on your device.
Note: Although this is a free app, you will be prompted to sign into your iTunes® account.
5. Type your Apple ID and password to complete the download process.

You are now ready to pair your hearing instruments with your iPhone, iPad, or iPod touch.

If you need assistance with your Apple device, please contact your local Apple store to make an appointment.